

VARIO DRAIN200
WOUND AND FLUID ACCUMULATION DRAINAGE SYSTEMS
SPRAL CUT CATHETER
WITHOUT PLUNGER


VARIO DRAIN200
WOUND AND FLUID ACCUMULATION DRAINAGE SYSTEMS
SPRAL CUT CATHETER

NO STERILISED

DISPOSABLE : DO NOT RESTERILISE



INSTRUCTIONS FOR USE

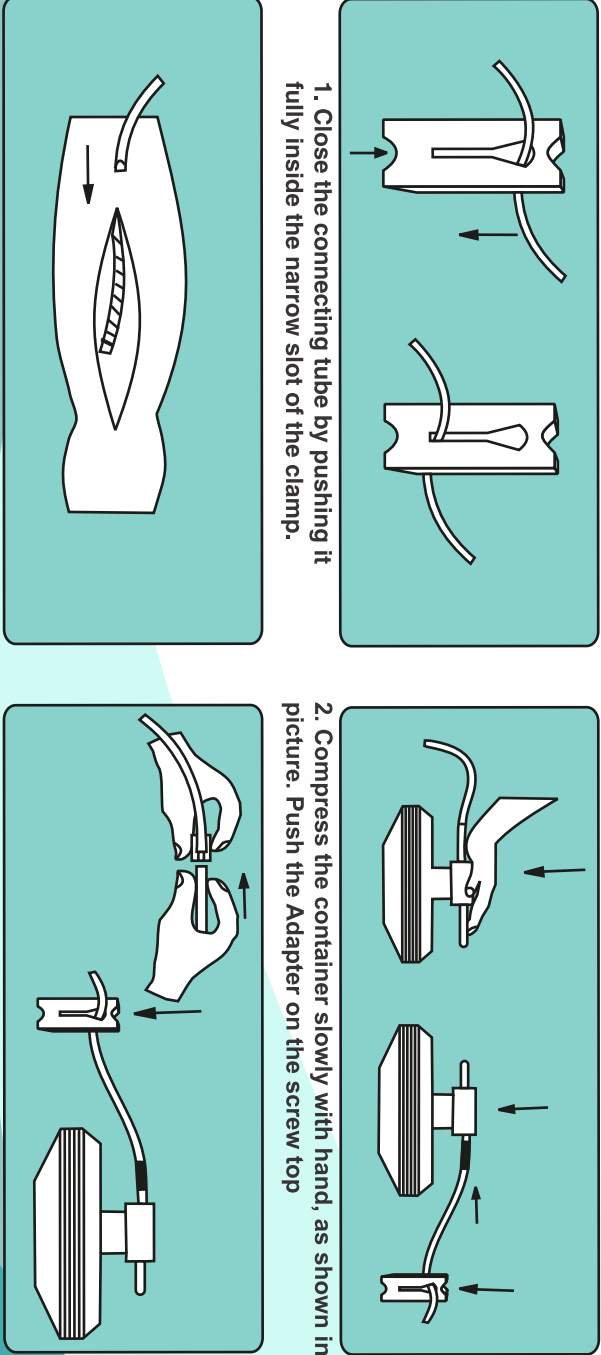


3. Place the spiral cut end of the catheter tube inside the wound upto the reference line and close the wound.

4. Insert the catheter tube into the connector. Remove the clamp and now suction is established.

WITH NEEDLE
WITHOUT PLUNGER

VARIO DRAIN200
WOUND AND FLUID ACCUMULATION DRAINAGE SYSTEMS
SPRAL CUT CATHETER

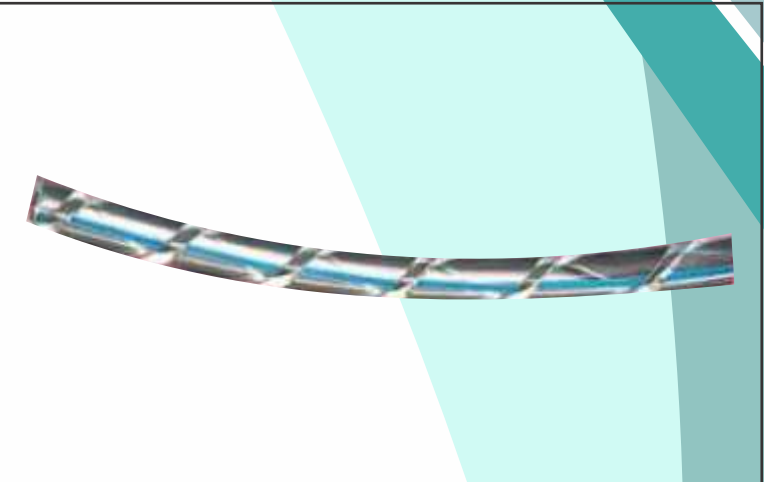

VARIO DRAIN200
WOUND AND FLUID ACCUMULATION DRAINAGE SYSTEMS
SPRAL CUT CATHETER

- Due to spiral cut absorption area is 4 times higher than conventional porous.
- Eliminates chances of blockage due to clots and ensures continuous drainage.
- Due to higher absorption surface the function is not limited by FG size of catheters.

PATENTED

VARIO DRAIN200
WOUND AND FLUID ACCUMULATION DRAINAGE SYSTEMS

Article No.	..
Size	..
Batch No.	..
Mfg. Date	..
Exp. Date	..
Max. Retail Price	..

MR Safe --Device made of non-metallic materials

CE 2460 EU REF


Manufactured by
DEVON INNOVATIONS (P) LTD.,
#27 A, Near State Bank of India, Electronic City,
Phase-1,Hosur Road, Bangalore - 560 100,
Tel :-+9180 28522354 / 367
Email :- sales@devoncoath.com,

Authorized by
Satumuststraat 46-62, Unit 032,
232 HB Heerdsburg, The Netherlands
Phone +31 23 5656337
Web: www.amstermed.nl
Mail: info@amstermed.nl

Net Qty:1No L-03 Rev.:09
Mfg.Lic.No:

VARIO DRAIN200
WOUND AND FLUID ACCUMULATION DRAINAGE SYSTEMS
SPRAL CUT CATHETER

WITH NEEDLE
WITHOUT PLUNGER

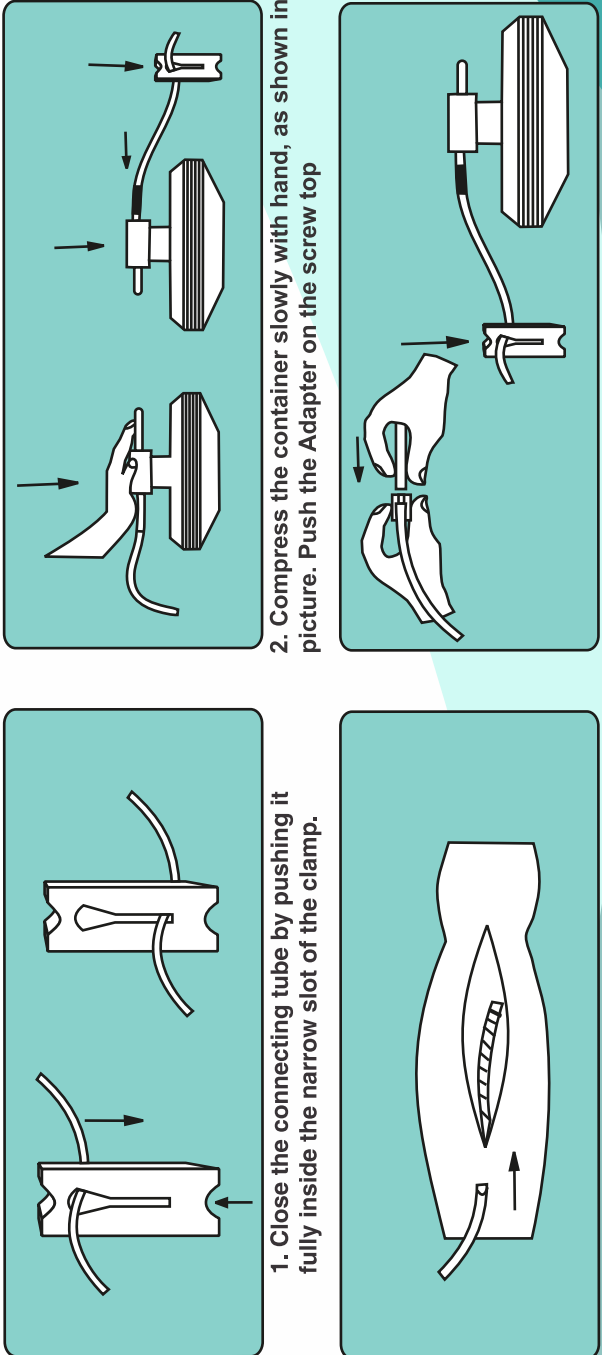

1. Close the connecting tube by pushing it fully inside the narrow slot of the clamp.

2. Compress the container slowly with hand, as shown in picture. Push the Adapter on the screw top

4. Insert the catheter tube into the connector. Remove the clamp and now suction is established.

INSTRUCTIONS FOR USE




DISPOSABLE : DO NOT RESTERILISE

VARIO DRAIN200
WOUND AND FLUID ACCUMULATION DRAINAGE SYSTEMS
SPRAL CUT CATHETER

NO STERILISED

VARIO DRAIN200
WOUND AND FLUID ACCUMULATION DRAINAGE SYSTEMS
SPRAL CUT CATHETER

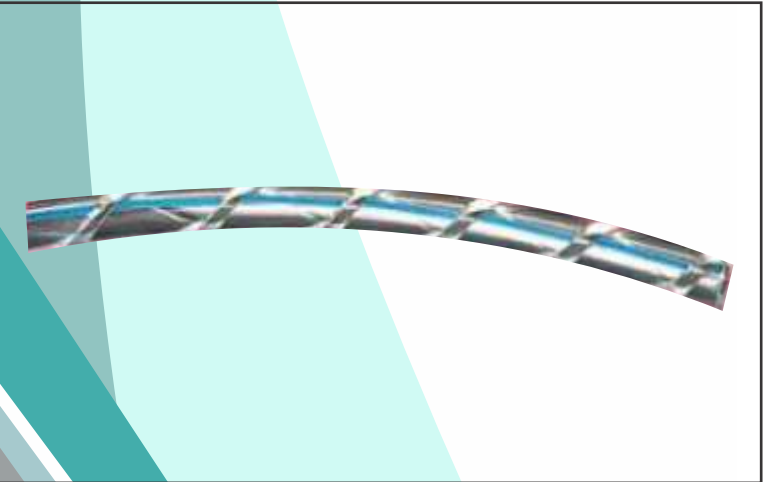

- Due to spiral cut absorption area is 4 times higher than conventional porous.
- Eliminates chances of blockage due to clots and ensures continuous drainage.
- Due to higher absorption surface the function is not limited by FG size of catheters.

PATENTED

VARIO DRAIN200
WOUND AND FLUID ACCUMULATION DRAINAGE SYSTEMS
SPRAL CUT CATHETER
WITHOUT PLUNGER